

- Clean before first and each use.
- Before first use and regularly, place in boiling water for 5 minutes. This is to ensure hygiene.
- Check nipple carefully before each use, especially when the child has teeth. Pull the nipple in all directions and throw away at the first signs of damage or weakness.
- Dishwasher safe in top rack (max. 149° F / 65° C). Food colorings may discolor components.
- Do not use abrasive cleaning agents. Only soft bristle brushes or soft sponges should be used.
- Disinfect using one of the following methods: steam sterilizing (electric or microwave according to manufacturers' instructions), boiling in water for at least 5 minutes or liquid sterilizing solution.
- Do not leave the nipple in direct sunlight or heat, or leave in sterilizing solution for longer than recommended, as this may affect the product function.
- Do not use pin or sharp object to enlarge feeding hole.
- Store nipple in a dry, sealed container.
- Insert nipple into top of bottle screw ring until you hear a "click".
- Use sugar free food or liquids only.
- NEVER heat or cool a sealed bottle. Remove the protective cap and nipple before heating, especially before microwave heating. Stir thoroughly after heating to evenly distribute heat.
- When first used and sometimes for later usages, the slit in the ventilation (the raised, slit bubble on the underside of the nipple base) can stick together. This will not allow air to enter the bottle during feeding and as a result, the nipple will collapse. Slowly brush over the bubble (ventilation) a few times with your thumb or fingers to re-open the slit.
- Bottle nipple fits all MAM wide neck bottles and trainers.

▲ CAUTION - for your child's safety and health!

- Always use this product with adult supervision.
- Never use feeding nipples as a soother.
- Continuous and prolonged sucking of fluids will cause tooth decay.
- Always check food temperature before feeding.